



MODEL #: **TP-7777**




Triple Play VT® – Universal Hip Orthosis


INSTRUCTIONS FOR USE

DESCRIPTION: Orthotic and prosthetic supply, accessory, and/or service component of another HCPS code. The orthosis has three (3) chambers that fill from the distal upward toward the proximal end of the extremity.


1. Remove optional gel pack and place in freezer for a minimum of 1 hour.
NOTE: Each time the gel wrap thaws, refreeze for 1 hour.
2. Remove gel from freezer, re-attach gel pack on the inside of the wrap.
3. Make sure to keep a protective thin cloth barrier between the skin and gel pack. Place the hip orthosis around the hip where needed. Secure the long strap around the waist and the shorter strap around the thigh.
4. Connect the hose to the AUX port on the Triple Play VT unit. Press and hold the Power button until the lights on the unit are illuminated.
5. Press the MODE button on the Triple Play VT until the blue AUX light is illuminated. To increase or decrease the inflation pressure of the wrap use the corresponding up or down arrows.
NOTE: Pressure cannot be altered during an inflation cycle.

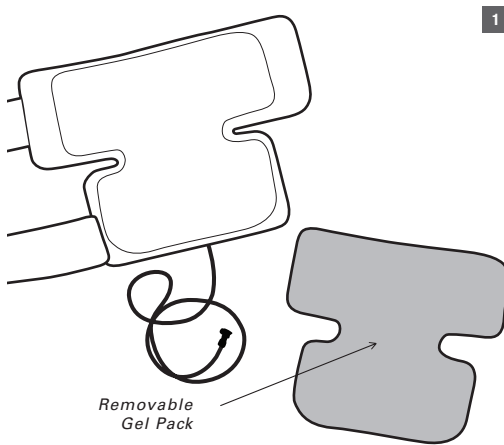


1

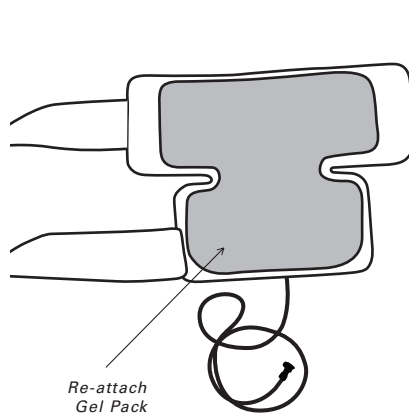


2

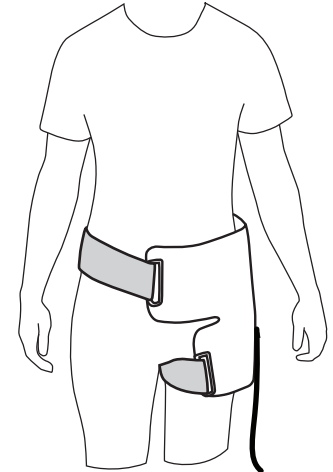

3

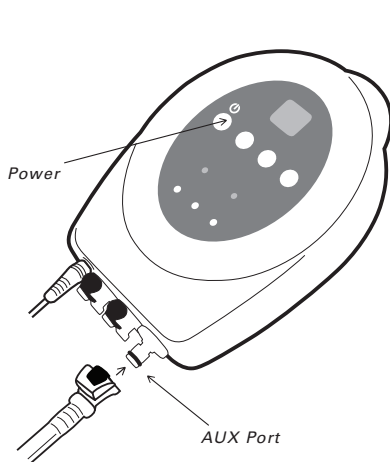

4

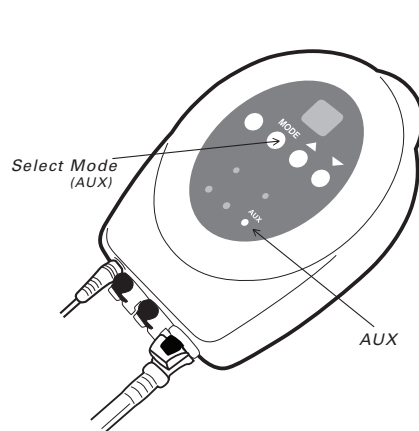

5

For questions about this product call: **800.994.0464**
www.CompressionSolutions.us

OPTIONAL HAND BULB

Your product is provided with an optional hand bulb which can be used to inflate/deflate the wrap in the absence of the Triple Play VT unit. If you desire, you may use scissors to cut the tubing just below the insertion into the wrap. Then place the barbed connector on the bulb into the tubing connected to the wrap.

