









MODEL #: **TP-4444**



Triple Play VT® – Universal Knee Orthosis INSTRUCTIONS FOR USE

DESCRIPTION: Knee orthosis, elastic with condylar pads and joints, with or without patellar control, prefabricated. The orthosis contains rigid metal stays or stabilizers to support the weakened body member. The orthosis has three (3) chambers that fill from the distal upward toward the proximal end of the extremity.

1. Remove optional gel pack and place in freezer for a minimum of 1 hour.
NOTE: Each time the gel wrap thaws, refreeze for 1 hour.
2. Remove the adjustable hinges from the inside of the brace. Secure these hinges back down to the brace in a location that will allow the condyle pads to rest securely on the inside and outside of the knee.
NOTE: For smaller knees, these hinges will be located closer to the center, for larger knees these hinges will be located closer to the edges.
3. Remove gel from freezer and secure the gel wrap on the inside of the brace making sure to not place it over the condyle pads.
4. Place a protective barrier between the gel pack and the skin. A cloth sleeve is included in your package. Any thin cloth barrier will be effective. Place the brace over the knee and secure the two straps around the knee to achieve a snug fit.
5. Connect the hose to the AUX port on the Triple Play VT unit. Press and hold the Power button until the lights on the unit are illuminated.
6. Press the MODE button on the Triple Play VT until the blue AUX light is illuminated. To increase or decrease the inflation pressure of the brace use the corresponding up or down arrows.
NOTE: Pressure cannot be altered during an inflation cycle.

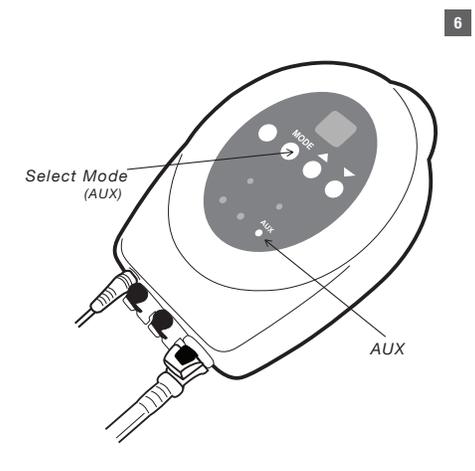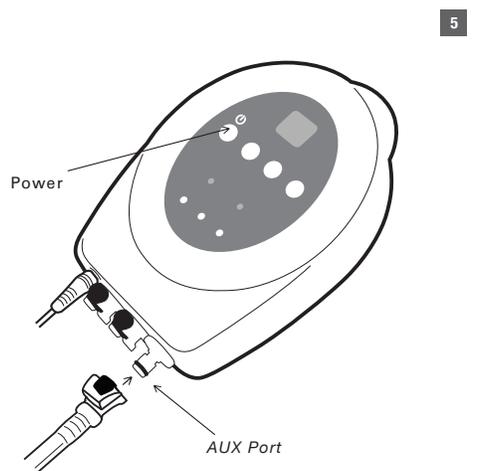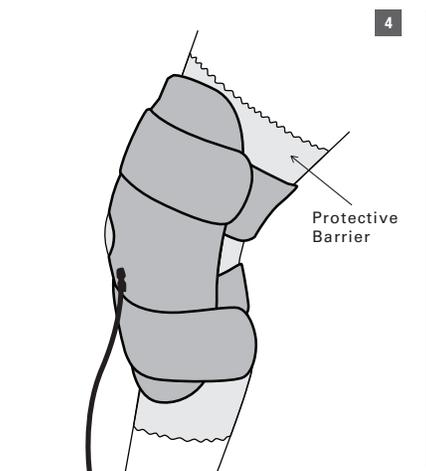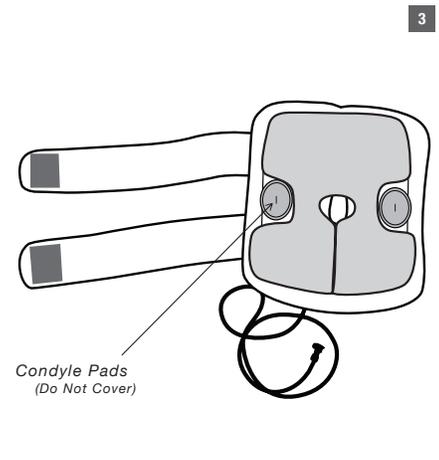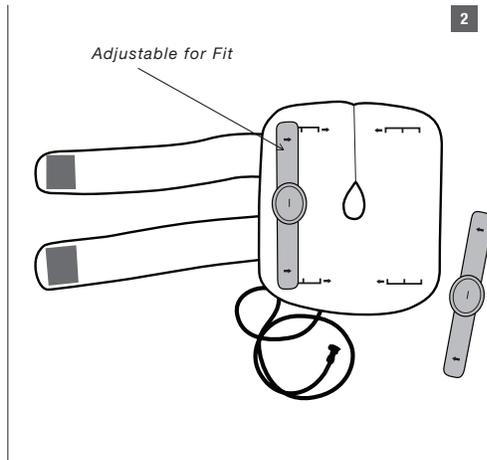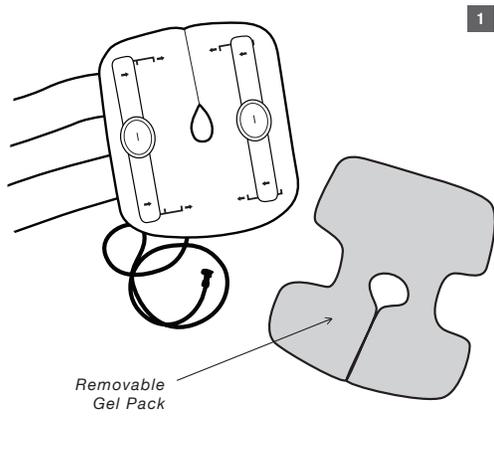

For questions about this product call: **800.994.0464**
www.CompressionSolutions.us

OPTIONAL HAND BULB

Your product is provided with an optional hand bulb which can be used to inflate/deflate the wrap in the absence of the Triple Play VT unit. If you desire, you may use scissors to cut the tubing just below the insertion into the wrap. Then place the barbed connector on the bulb into the tubing connected to the wrap.

